




Reliant

STENT GRAFT BALLOON CATHETER

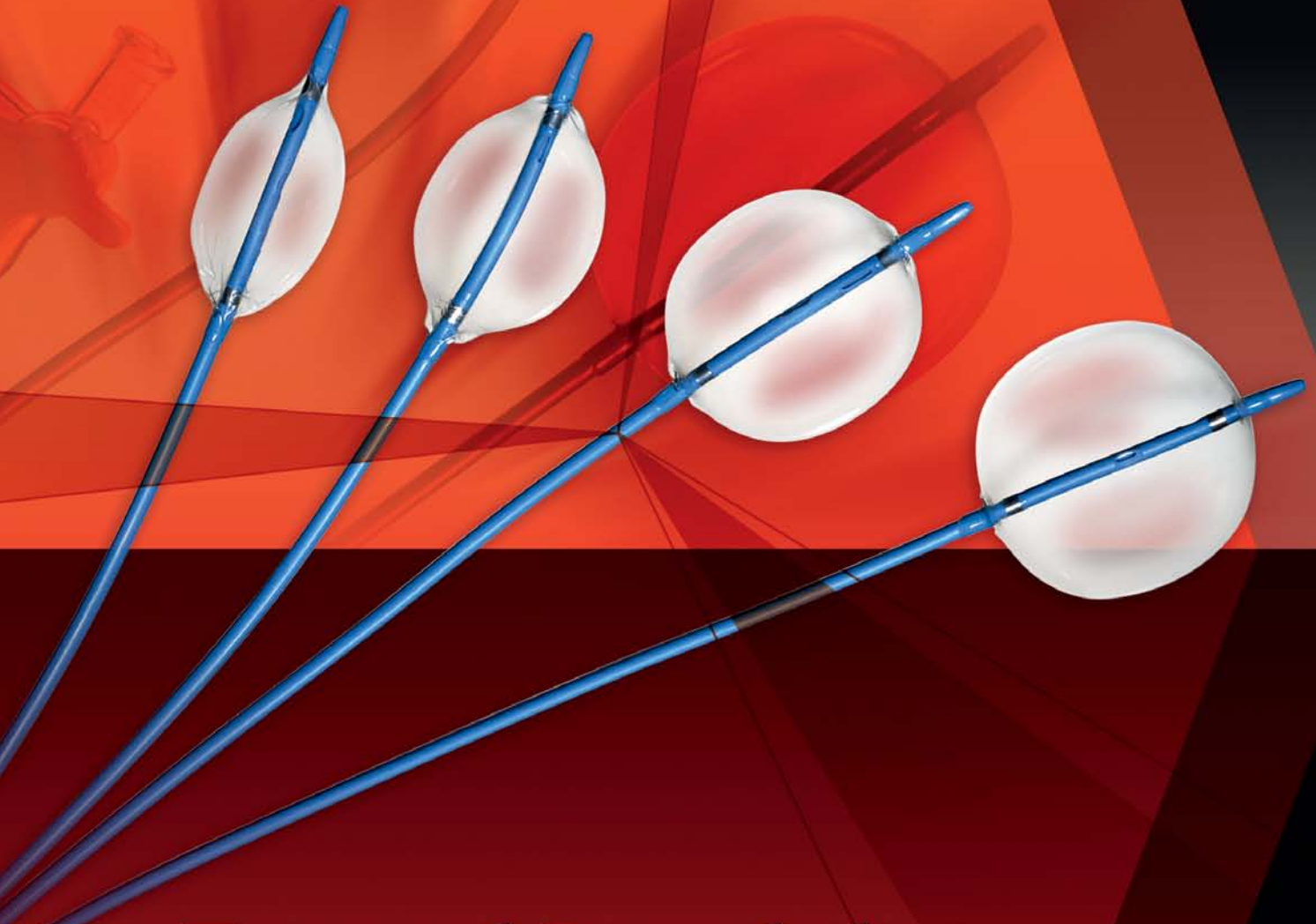


Expand Possibilities

➤ A single-solution balloon catheter for your stent graft procedure needs. **Reliant Stent Graft Balloon** is the versatile, reliable and comfortable solution that extends procedural possibilities every day.

Take the next step at medtronicendovascular.com

Innovating for life.

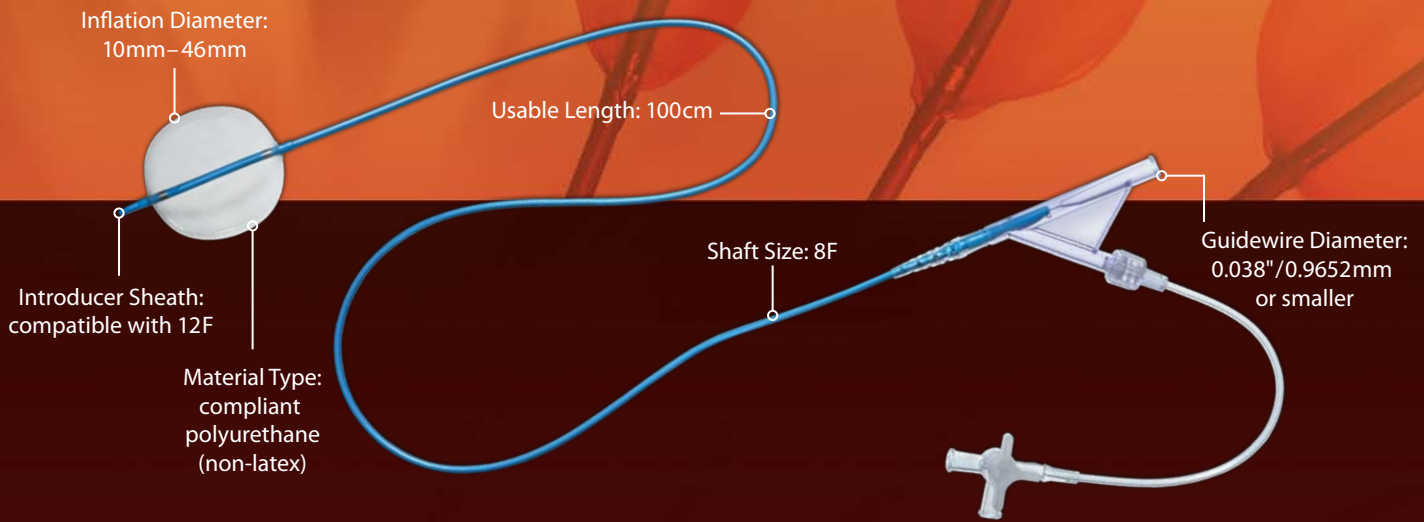

MULTIPLE PURPOSES, SINGLE SOLUTION

Clinical uses include:

- Abdominal and thoracic use
- Endograft modeling
- Endoleak sealing support

WIDEST RANGE OF BALLOON INFLATION DIAMETERS

Balloon inflation table

46mm Balloon	
Diameter (mm)	ml (cc)
10	3
20	9
30	19
40	41
46 ³	60

CAUTION: This table is only a guide. Balloon expansion should be carefully monitored under fluoroscopy. Do not exceed maximum inflation diameter (46mm). Rupture of balloon may occur.

PRODUCT INFORMATION

Reliant stent graft balloon catheter⁴

Product Code	Inflation Diameter (mm)	Shaft Size (F)	Usable Length (cm)	Sheath Compatibility (F)
AB46	10–46	8	100	12

Please reference appropriate product *Instructions for Use* for a more detailed list of intended use, warnings, precautions and potential adverse events.

References

1. Reliant Stent Graft Balloon Catheter *Instructions For Use*, Cook Coda Balloon Catheter *Instructions For Use*, Gore Tri-Lobe *Instructions for Use*.
2. Data on file with Medtronic, Inc. Maximum of 20 inflations, deflations.
3. Maximum inflation diameter.
4. Does not contain latex.

➔ www.medtronic.eu
www.medtronicendovascular.com

Medtronic Vascular, Inc.
 3576 Unocal Place
 Santa Rosa, CA 95403
 USA
 Tel: +1.707.525.0111

Medtronic International Trading Sàrl
 Route du Molliau 31
 CH-1131 Tolochenaz
 Switzerland
 Tel: +41.21.802.7000

Medtronic International Ltd.
 49 Changi South
 Avenue 2
 Singapore 486056
 Tel: +65.6436.5000

Medtronic of Canada Ltd.
 99 Hereford Street
 Brampton, Ontario L6Y 0R3
 Canada
 Tel: +1.905.460.3800

Medtronic Nederland
 Earl Bakkenstraat 10
 6422 PJ Heerlen
 The Netherlands
 Tel: +31.45.566.8000


Medtronic Australasia Pty Ltd.
 97 Waterloo Road
 North Ryde, NSW 2113
 Australia
 Tel: +61.29857.9000

Medtronic Latin America
 3750 NW 87 Avenue, Suite 700
 Miami, FL 33178
 USA
 Tel: +1.305.500.9328

Medtronic Ireland
 Parkmore Business Park West
 Galway
 Ireland
 Tel: +353.91.708000


Versatile

- For use in thoracic and abdominal endograft procedures
- Widest range of inflation diameters (10mm–46mm)¹
- Lowest crossing profile stent graft modeling balloon¹
- Compatible with 12F sheath

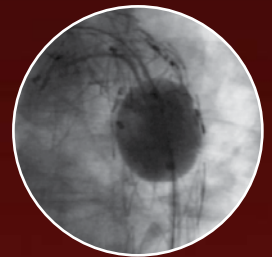

Reliable

- Consistent inflation and deflation time
- Stable expansion with minimum balloon overhang to reduce risk of vessel trauma
- Dependable expansion even after multiple inflations and deflations²

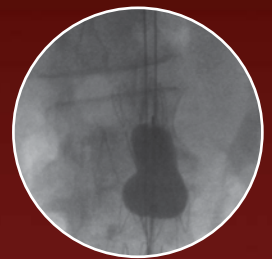

Conformable

- Improves conformability to vessel wall
- Expands kinks and smooths creases from the graft material
- Compliant balloon expansion

